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ENGLISH

Intended use:

The cranial fixation pins (adult or pediatric) are a reusable product whose purpose is to fix the patient's head during a surgical procedure. They are used in neurosurgical procedures.

Procedure:

1. Send the pins to the material sterilization center;
2. Open the sterilization kit;
3. Remove the pins from the kit;
4. Hold the pins by their protective covering and place them in the cranial fixation device;
5. Carefully remove the protective covering;
6. Position the patient's head near the cranial fixation device;
7. Carefully press the tip of the pin against the patient's head;
8. Perform the surgical procedure;
9. Remove the pins from the patient's head;
10. Place the protective covering back onto the pins;
11. Remove the pins from the cranial fixation device;
12. Send the pins to be cleaned;
13. Store the pins in the sterilization kit.

Precautions | Warnings:

- For more information the associated equipment used, consult the user manual for said equipment.
- Cranial fixation pins are **not provided sterile**, they must therefore be cleaned and sterilized prior to use.
- Avoid any action that could deform the product, since any deformity can compromise its sterility and functionality.
- In the event any kind of anomaly is found on the tip of the fixation pin, it should be discarded according to local laws.
- Pins should only be used with Micromar Cranial Fixation Systems. User is fully liable for the use of any unauthorized piece, part or accessory.
- It is recommended that pediatric pins be used for any surgical procedure done on children between the ages of 3 and 9. Nevertheless, the surgeon must assess the patient's conditions, including consideration of age, prior to planning the surgery.
- Additional care is needed in relation to pressure used for cranial fixation done on babies, children or the elderly or on an unhealthy bone, since bones in this case are more fragile than a healthy or adult bone.
- Whenever possible, avoid positioning the pins on irregular or fragile bones, abnormally thin bones or in areas with different bone consistencies and thicknesses, near the orbit, the temporal fossa, the venous sinuses and areas near previous craniotomies. The surgeon should consider the cranial conditions, including thickness, prior to planning the surgery.
- This product may only be manipulated by a surgeon specialized in neurosurgery with experience performing cranial fixation.
- Any failure to follow these instructions and the manual for any equipment used in association can cause the product to be used incorrectly and represents misuse, which poses a potential risk of patient death.
- Pins have a piercing tip and should therefore be handled with care.
- Only remove the protective covering from the pin after it has been fully placed in the cranial fixation device, avoiding damage to the pin tips and preventing any perforation of the surgical glove and of the operator's hand.
- After the procedure is over, the pins must be sent for cleaning.
- After cleaning, the components should be immediately dried using a dry compress and compressed air.
- Items exposed to Creutzfeldt-Jakob disease should be discarded according to local regulations.
- Equipment used to treat patients suffering from Creutzfeldt-Jakob disease MUST NOT BE reused for other patients.

Contraindications:

Do not use the product in the risk situations established by the surgeon.

Side effect:

No side effects were found from use of the product when handled by a skilled professional and in compliance with the guidelines of these instructions.

Manual cleaning (immersion):

Immerse the disassembled parts in a container with cold to warm water solution (up to 40 ° C) with enzymatic detergent (dilution as recommended by the manufacturer). Leave the solution to act for at least 15 minutes. With a soft brush remove the residues of the items, paying attention to the recesses and holes. Rinse with running water. Dry with surgical compress or wiper and clean compressed air.

Automatic cleaning (ultrasound):

Immerse the disassembled parts in an ultrasonic cleaner vat with cold to warm water solution (up to 40°C) with enzymatic detergent (dilution as recommended by the manufacturer). Activate the ultrasound for at least 15 minutes. Rinse with running water. Dry with surgical compress or wiper and clean compressed air.

Useful life time:

Cranial fixation pins have a useful life of 5 years as of the date of issue of the invoice.

Disposal

Pins are piercing. Discard the product according to local laws.

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