Símbolos / Symbols / Símbolos

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ENGLISH

Product Description: The Disposable Reflective Sphere for Navigation is a single-use product, the sphere is made of ABS, its surface is covered with a coating of microspheres that uniformly reflects infrared light. It has a set of four, five and eight spheres, used in image-guided surgical navigation procedures.

Intended Use: The Disposable Reflective Sphere for Navigation is a single use device which aims to reflect the infrared light, allowing the instruments, to which they are coupled, to be tracked during surgical image-guided navigation procedures.

Purpose and Working Principle: The Disposable Reflective Sphere for Navigation is used in conjunction with a Surgical Navigator, which is dedicated to guiding surgeons, through images, during surgical procedures. It allows surgical instruments to be spatially tracked and visualized in sync with the patient's anatomical structures in real time. The technique, known as Image-Guided Surgery, consists of having the patient's anatomy and surgical instruments reproduced through graphic-computational reconstructions using monitors.

The Disposable Reflective Sphere for Navigation is a communication interface between the surgical instruments and the navigation station. The tracking sensor detects and uses the spheres to calculate the geometric position of the corresponding instruments.

 Caution:

The Disposable Reflective Sphere for Navigation is intended for specialized procedures, which must be performed by duly qualified professionals. The product must be used in a surgical environment and under conditions suitable for the health and safety of the patient.

How to Use Disposable Reflective Sphere for Navigation:

1. Open the surgical grade packaging;
2. Visually check the primary packaging (blister sealed with Tyvek);
3. Open the blister;
4. Remove the spheres from the blister;
5. Connect the spheres to the instruments, under pressure or screwing them (according to the model);
6. Accomplish the surgical procedure;
7. Remove the spheres from the instruments;
8. Discard the spheres.

Cautions / Warnings:

- Before using the product, check the packaging regarding its integrity in order to ensure that the product sterility is not compromised.
- Do not use the product if the sterilization packaging is damaged.
- Check the expiration date of the product before opening the packaging.
- Avoid any action that will deform the product, because it can compromise its sterility and its operation.
- This product is intended to be used only in surgical center.
- The reflective spheres must only be used by a physician specialized in neurosurgery or orthopedic surgery of the column, with experience in navigation surgery.
- The spheres placed under pressure must only be used with instruments that have a pin fit and the threaded spheres must only be used with instruments that have a thread fit. The use of any part, accessory or material not specified is entirely the user's responsibility.
- The product is supplied sterilized by Ethylene Oxide (EO).
- The reflective spheres are single use devices, therefore, it is forbidden to reprocess and/or re-sterilize it.
- Make sure that the area between the Navigator tracking sensor and the reflective spheres is not obstructed during the entire procedure.
- Make sure that the reflective spheres are firmly connected to the surgical instruments pins (without movement) and there is not gap between the spheres and the base of the pins.
- If there is splash of substances in the spheres, replace them for clean spheres. Otherwise, the navigation can be compromised.
- All the spheres must be with their reflective surface in good condition. If it is damaged, replace the sphere.
- Replace the gloves after handling the spheres.
- Failure to follow this instruction for use and the associated use equipment manual can lead to incorrect use of the device and therefore misuse, which increases the risk of death to the patient.

Contraindications: Do not use the product in the risk situations foreseen by the surgeon.

Adverse effects: No side effect was found when the product is used by a specialized professional following this instruction for use.

Disposal : The spheres have glass microspheres. Dispose of the product in accordance with current hospital regulations and local legislation.

Traceability: All Micromar products have sequential batches that allow traceability, thus promoting greater safety for professionals qualified for the procedure. Through this batch number it is possible to know the entire history of the product from the manufacturing process to the moment of distribution

 Presentation and Sterilization Method:

The Disposable Reflective Sphere for Navigation is supplied sterile by Ethylene Oxide (ETO) and for single use with the following packaging:

- Primary packaging: blister made of heat-sealed PET with Tyvek containing the spheres;
- Secondary packaging: surgical grade paper + PET/PP film;
- Tertiary packaging: triplex paper box.

Expiration date: The shelf life according to the stability study is 4 years; An expiration date once opened does not apply.

Additional Information: Single-use product. If an incident caused by the product occurs, the professional must immediately inform the manufacturer.

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