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ENGLISH

Product Description: The Disposable Pin for Cranial Fixation is a single-use product, its composition is aluminum, available in different sizes and sets. It is used in neurosurgical procedures.

Intended Use: The Disposable Pin for Cranial Fixation is a single use device which aims to enable the fixation of the patient head during surgical procedures. It is used in neurosurgical procedures.

Purpose and Working Principle: The Disposable Pin for Cranial Fixation, when attached to the Micromar Stereotaxic System, is pressed against the points established on the patient's head until it is fixed to the cranial bone.

 Caution:

The Disposable Pin for Cranial Fixation is intended for specialized procedures, which must be performed by duly qualified professionals. The product must be used in a surgical environment and under conditions suitable for the health and safety of the patient.

How to use The Disposable Pin for Cranial Fixation:

1. Open the surgical grade packaging;
2. Visually check the primary packaging (blister sealed with Tyvek);
3. Open the blister;
4. Remove the pins from the blister;
5. Hold the pins by the protective cover and mount them in the cranial fixation system;
6. Remove the protective cover carefully;
7. Position the patient head next to the cranial fixation system;
8. Press carefully the tip of the pin against the patient head until it reaches the skull bone;
9. Accomplish the surgical procedure;
10. Remove the pins from the patient head;
11. Put the protective covers on the pins again;
12. Remove the pins from the cranial fixation system;
13. Discard the pins.

Cautions / Warnings:

- Before using the product, check the packaging regarding its integrity in order to ensure that the product sterility is not compromised.
- Do not use the product if the sterilization packaging is damaged.
- Check the expiration date of the product before opening the packaging.
- Avoid any action that will deform the product, because it can compromise its sterility and its operation.
- This product is intended to be used only in surgical center.
- The pins must be used with Micromar HeadFix. The use of any part, accessory or material not specified is entirely the user's responsibility.
- If the procedure is accomplished in children between 3 and 9 years old, it is recommended the use of the pediatric pins. Nevertheless, the physician must evaluate the patient's condition, considering his / her age, before planning the surgery.
- It is necessary a greater care regarding the pressure during the skull fixation of babies, children, elderly or in sick bone, once it is more fragile than an adult and healthy bone.
- When possible, avoid positioning the pins over uneven or fragile bone, abnormally thin bone or in areas of varying bone consistencies and thickness, near the orbit, the temporal fossa, venous sinuses and prior craniotomy bone flap sites. The physician must consider the skull condition, including its thickness, before planning the surgery.
- The pins have a sharp tip, therefore they must be handled carefully.
- This product must only be used by a physician specialized in neurosurgery or with experience in cranial fixation.
- The product is supplied sterilized by Ethylene Oxide (EO).
- This product is a single use device, therefore, it is forbidden to reprocess and/or re-sterilize it.
- Failure to follow this instruction for use and the associated use equipment manual can lead to incorrect use of the device and therefore misuse, which increases the risk of death to the patient.
- Only remove the pin protective cover after its total fit in the cranial fixation system, avoiding the damage of the pins tip and the perforation of the surgical glove and the user hand.

Contraindications: Do not use the product in the risk situations foreseen by the surgeon.

Adverse Effects: No side effect was found when the product is used by a specialized professional following this instruction for use.

Disposal : The pins are sharp pointed. Dispose the product according to local sanitary regulations. Dispose of the product in accordance with current hospital regulations and local legislation.

Traceability: All Micromar products have sequential batches that allow traceability, thus promoting greater safety for professionals qualified for the procedure. Through this batch number it is possible to know the entire history of the product from the manufacturing process to the moment of distribution.

Storage: The Disposable Pin for Cranial Fixation must be stored in a dry and cool place, at a temperature of 20 to 30° C, with relative humidity between 55 to 75%. It must be protected from direct solar radiation, in its original, unopened packaging and must not be damaged.

 Presentation and Sterilization Method:

The Disposable Pin for Cranial Fixation is supplied sterile by Ethylene Oxide (ETO) and for single use with the following packaging:

- Primary packaging: blister made of heat-sealed PET with Tyvek containing 03 pins.
- Secondary packaging: surgical grade paper + PET/PP film;
- Tertiary packaging: triplex paper box.

Expiration date: The shelf life according to the stability study is 4 years; An expiration date once opened does not apply.

Additional information: Single-use product. If an incident caused by the product occurs, the professional must immediately inform the manufacturer.

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