HeadFix®

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Português · English · Español

MICROMAR

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PRODUCT DESCRIPTION:

HEADFIX CRANIAL FIXATION SYSTEM

USE THE PRODUCT ONLY AFTER READING THE INSTRUCTIONS AND THEIR RESPECTIVE WARNINGS

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NACIONAL@MICROMAR.COM

WWW.MICROMAR.COM

MICROMAR IND. COM. RESERVES THE RIGHT TO MAKE CHANGES TO THIS MANUAL WITHOUT PRIOR NOTICE.

CONTACT OUR SALES DEPARTMENT TO RECEIVE THE LATEST VERSION OF THE USER MANUAL.

THIS MANUAL CONTAINS INFORMATION PROTECTED BY COPYRIGHT.

THIS MANUAL MAY NOT BE REPRODUCED, TRANSLATED OR TRANSMITTED BY ANY ELECTRONIC OR OTHER MEANS, IN WHOLE OR IN PART, WITHOUT EXPRESS AUTHORIZATION FROM MICROMAR.

ÍNDICE

1. INTRODUCTION.....	41
2. ILLUSTRATIONS.....	41
3. INDICATIONS FOR USE.....	41
4. CONTRAINDICATIONS.....	41
5. SYSTEM.....	42
5.1. COMPONENTS LIST	42
6. BASE	43
6.1. HEADFIX BASIC BASE - HF041000FAA.....	43
6.1.1. CHECKING THE STABILITY OF THE HEADFIX BASIC BASE SET - HF041000FAA BEFORE THE SURGI- CAL PROCEDURE	45
6.1.2. ASSEMBLING THE HEADFIX BASIC BASE SET - HF041000FAA ON THE SURGICAL TABLE	46
6.2. HEADFIX BASE - HF051000FAA.....	48
6.2.1. CHECKING THE STABILITY OF THE HEADFIX BASE SET - HF051000FAA BEFORE THE SURGICAL PRO- CEDURE	49
6.2.2. ASSEMBLING THE HEADFIX BASE SET - HF051000FAA ON THE SURGICAL TABLE	50
7. ARTICULATED SYSTEM.....	53
7.1. HEADFIX BASIC ARTICULATED SYSTEM - HF042000FAA.....	53
7.1.1. ASSEMBLING THE HEADFIX BASIC ARTICULATED SYSTEM - HF042000FAA ON THE BASE SET.....	54
7.2. HEADFIX ARTICULATED SYSTEM - HF052000FAA.....	55
7.2.1 ASSEMBLING THE HEADFIX ARTICULATED SYSTEM - HF052000FAA ON THE BASE SET.....	56
8. SKULL CLAMP.....	57
8.1. HEADFIX BASIC SKULL CLAMP - HF043000FAA	58
8.1.1. POSITIONING THE HEADFIX BASIC SKULL CLAMP - HF043000FAA ON THE PATIENT'S HEAD.....	59
8.1.2. STABILIZING THE PATIENT'S HEAD.....	60
8.2. HEADFIX SKULL CLAMP - HF053000FAA	62
8.2.1. POSITIONING THE HEADFIX SKULL CLAMP - HF053000FAA ON THE PATIENT'S HEAD.....	63
8.2.2. STABILIZING THE PATIENT'S HEAD.....	64
8.3. ASSEMBLY OF FIXATION PINS ON THE SKULL CLAMP.....	66
8.3.1. PREPARATION FOR USE OF FIXATION PIN - ADULT MODEL	66
8.3.2. PREPARATION FOR USE OF FIXATION PIN - PEDIATRIC MODEL.....	67
8.4. SEMIARC.....	67
8.4.1. SEMIARC ADJUSTMENT	68
8.5. DYNAMOMETER.....	69
8.5.1. DYNAMOMETER FUNCTIONING.....	69
9. ADJUSTMENT OF LEVER PRESSURE.....	70
10. DISASSEMBLY.....	71
11. CLEANING.....	72
12. PREVENTIVE AND CORRECTIVE MAINTENANCE	72
13. SYMBOLS.....	73
14. TRANSPORT AND STORAGE.....	73
15. WARNINGS AND PRECAUTIONS.....	74

1. INTRODUCTION

This manual aims to help the user operate the HeadFix Cranial Fixation System safely and effectively. Before using the equipment, the user must carefully read and fully understand the manual. This manual should be kept with the equipment for reference in case of doubts.

2. ILLUSTRATIONS

The images in this manual present the product for illustrative purposes. Certain differences may occur compared to the physical components. These differences in the illustrations do not affect the use of the product.

3. INDICATIONS FOR USE

The HeadFix Cranial Fixation System is intended for use by physicians when they need to support and fix the patient's head during neurosurgical procedures.

The product described in this manual can only be used in combination with other equipment or components expressly recognized by the Manufacturer as compatible.

Compatible products:

- HeadFix HF03B Cranial Fixation;
- Accessories for cranial fixation - HF03B;
- Micromar Stereotactic TM03B;
- HeadFix HF02B Cranial Fixation;
- Accessories for cranial fixation - HF02B
- AimNav instruments - Cranial Toolbox - NN00100FAA;
- Disposable Pin for Cranial Fixation (Aluminum);
- Disposable Pin for Cranial Fixation (Titanium);
- Disposable Pin for Cranial Fixation - Stainless steel;
- Reusable Cranial Fixation Pins

4. CONTRAINDICATIONS

Do not use the product in the risk situations foreseen by the surgeon.

5. SYSTEM

The HeadFix Cranial Fixation System is a set of interchangeable components that, when assembled, enable the fixation and support of the patient's head according to the surgeon's needs during neurosurgical procedures.

5.1. COMPONENTS LIST

The system is composed of 6 sets.

ITEM	DESCRIÇÃO	CÓDIGO
(A)	HeadFix Basic Base	HF041000FAA
(B)	HeadFix Base	HF051000FAA
(C)	HeadFix Basic Articulated System	HF042000FAA
(D)	HeadFix Articulated System	HF052000FAA
(E)	HeadFix Basic Skull Clamp	HF043000FAA
(F)	HeadFix Skull Clamp	HF053000FAA

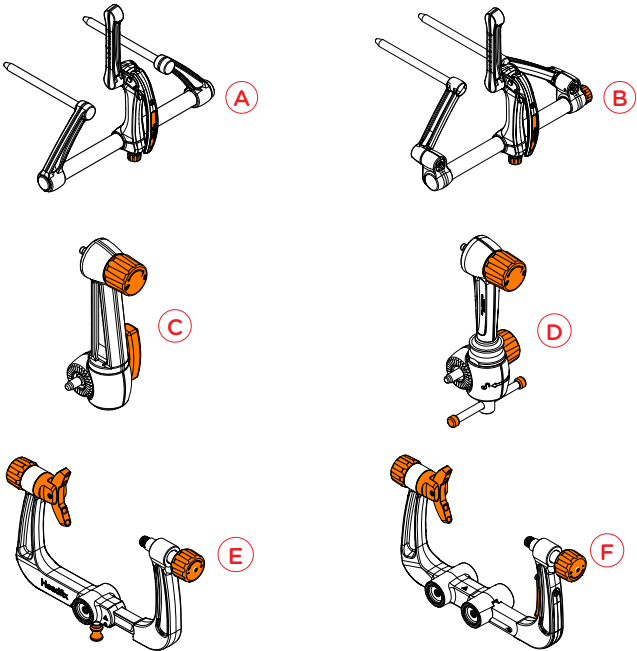
Figure 1

6. BASE

The Base is a set that allows the cranial fixator to be attached to surgical tables, enabling height adjustment and lateral movement relative to the table. It is available in two models:




ITEM		MODEL
		HF041000FAA HEADFIX BASIC BASE
		HF051000FAA HEADFIX BASE

Figure 2

NOTE: Both are compatible with the HeadFix Basic Articulated System (HF042000FAA) and the HeadFix Articulated System (HF052000FAA).

6.1. HEADFIX BASIC BASE — HF041000FAA

The HeadFix Basic Base allows adjusting the center-to-center distance of the axes to fit surgical tables, with the distances shown in Figure 3

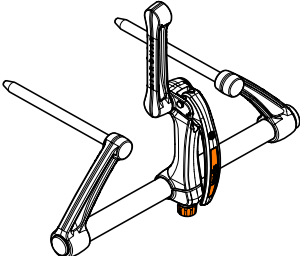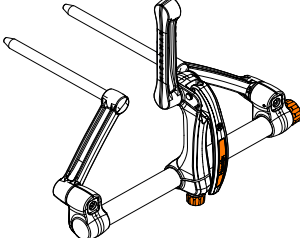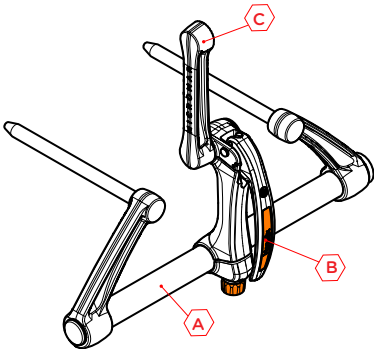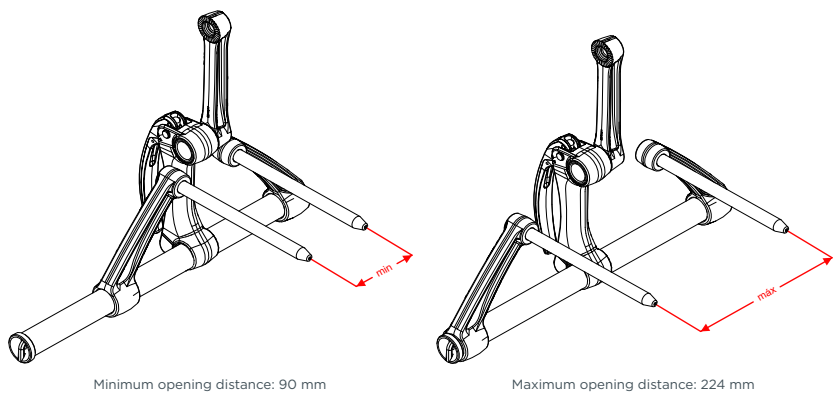
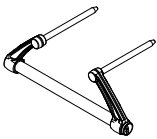
Figure 3

⚠ WARNING: Before starting the surgical procedure, ensure that the distances provided by the Head-Fix Basic Base (maximum and minimum) are compatible with the surgical table to be used.

⚠ WARNING: For information about the surgical table, please refer to the user manual provided by the table manufacturer.

The HeadFix Basic Base — HF041000FAA is composed of 3 sets

	ITEM	DESCRIPTION	QTY.
(A)		Surgical table adaptor	1
(B)		Quick Lock lever	1
(C)		Stretcher	1

6.1.1. CHECKING THE STABILITY OF THE HEADFIX BASIC BASE SET - HF-041000FAA BEFORE THE SURGICAL PROCEDURE

1. Place the HeadFix Basic Base assembly on a flat and clean surface **I**, and then position the assembly at an angle of approximately -45° relative to the supporting surface **II**.

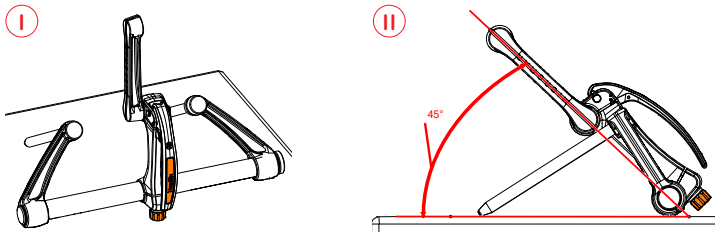
Figure 4

2. Keep the assembly positioned at approximately -45° relative to the supporting surface, then push the lever **a** to lock the entire assembly.

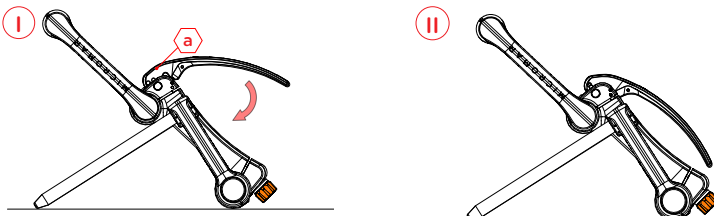
Figure 5

3. With the entire set locked, apply pressure to the Stretcher set using the palm of your hand **b**, to verify if the pressure of the lever is sufficient to guarantee the set's stability;

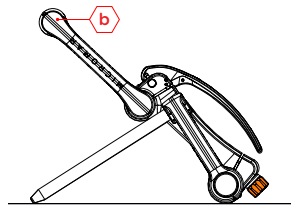
Figure 6

⚠ WARNING: During the stability test, there must be no movement whatsoever. If any movement occurs, repeat the procedure to adjust the lever pressure.

⚠ WARNING: The lever must never be pressed if the Stretcher set is not inserted inside the hole, since it will then damage the working of the Cranial Fixation System.

⚠ WARNING: The pressure of the set comes factory-adjusted.

NOTE: See chapter 9 for more information how to adjust the lever pressure.

6.1.2. ASSEMBLING THE HEADFIX BASIC BASE SET - HF041000FAA ON THE SURGICAL TABLE

1. First, remove the Allen key from the base;

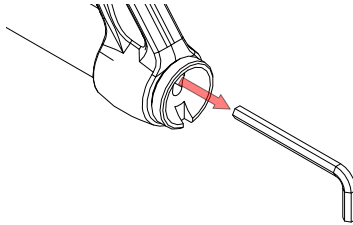
Figure 7

2. Insert the Allen key into the connection locking screw (right) **I**, then turn the Allen key counter-clockwise until the arm adjustment movement is released **II**;

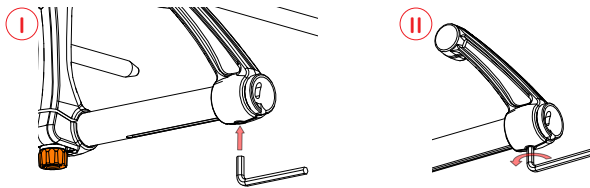
Figure 8

3. Adjust the distance between the centers of the axes according to the distance between the centers of the surgical table connections that will be used for the procedure;

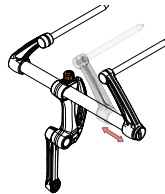
Figure 9

4. After adjusting the distance between the axle centers, lock the arm adjustment mechanism by turning the hex key clockwise;

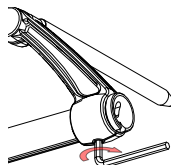
Figure 10

5. Return the Allen key to the base;

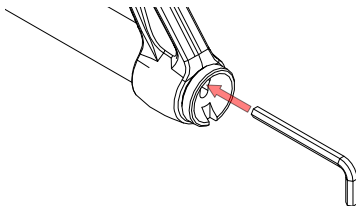
Figure 11

6. Pull the lever **I** and leave it unlocked **II**;

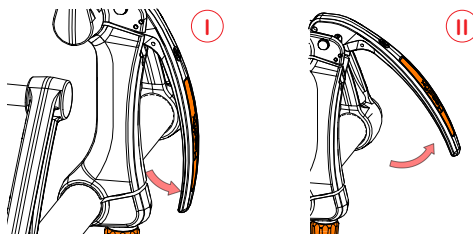
Figure 12

7. After the base assembly has all the joints released, leave it pointing toward the floor in order to facilitate the fixation of the articulated system by the user;

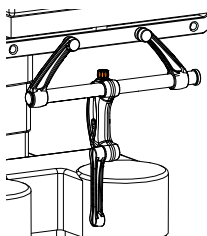
Figure 13

8. Lock the base using the surgical table's fixation mechanism.

6.2. HEADFIX BASE - HF051000FAA

The Headfix Base allows adjustment of the center-to-center distance between the axes for fitting onto surgical tables at a distance as illustrated in Figure 14.

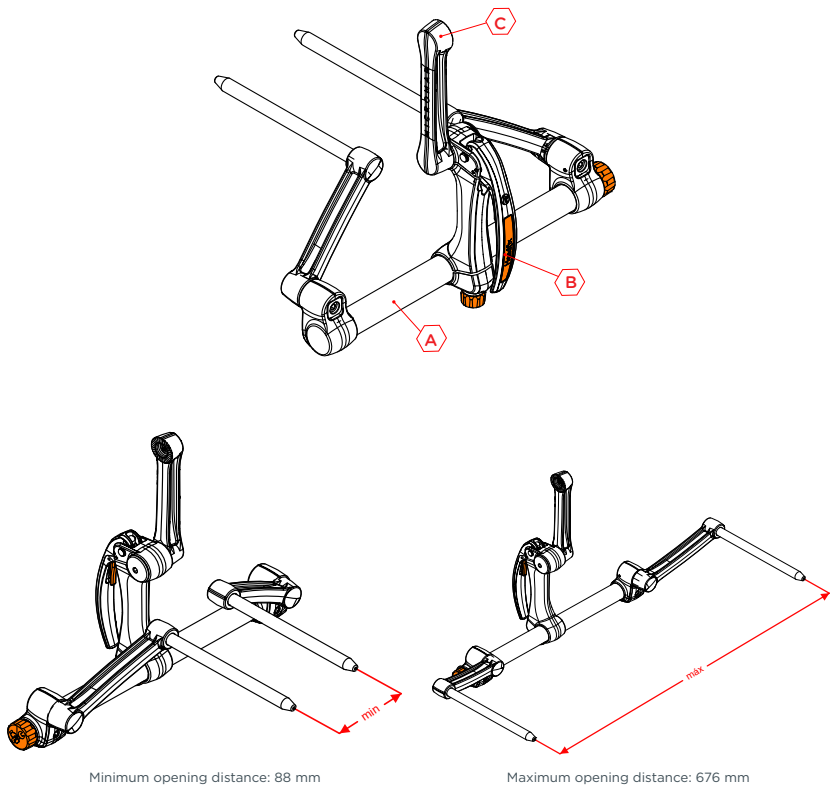
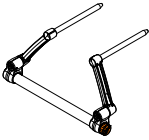
Figure 14

⚠ WARNING: Before starting the surgical procedure, ensure that the distances provided by the HeadFix Basic Base (maximum and minimum) are compatible with the surgical table to be used.

⚠ WARNING: For information about the surgical table, please refer to the user manual provided by the table manufacturer.

The HeadFix Base - HF051000FAA consists of 3 assemblies:

	ITEM	DESCRIPTION	QTY.
A		Articulated operating table adapter	1
B		Lever (quick-lock) with safety trigger	1
C		Stretcher	1

6.2.1. CHECKING THE STABILITY OF THE HEADFIX BASE SET - HF051000FAA BEFORE THE SURGICAL PROCEDURE

1. Place the HeadFix Base assembly on a flat and clean surface **I**, and then position the assembly at an angle of approximately -45° relative to the supporting surface **II**.

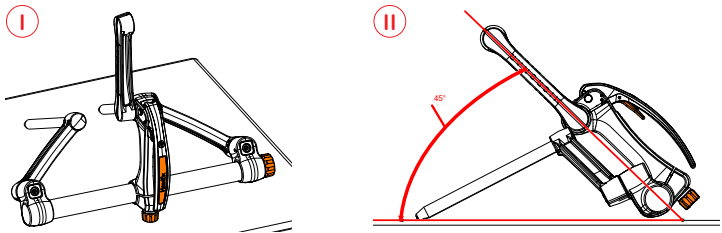
Figure 15

2. Keep the assembly positioned at approximately -45° relative to the supporting surface, then push the lever **a** to lock the entire assembly;

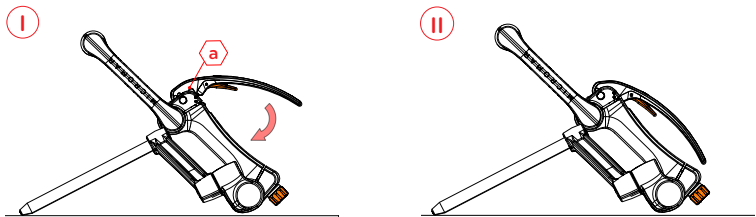
Figure 16

3. With the entire set locked, apply pressure to the Stretcher set using the palm of your hand **b**, to verify if the pressure of the lever is sufficient to guarantee the set's stability;

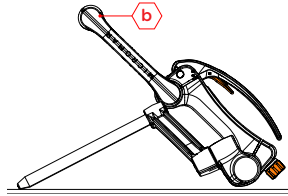
Figure 17

⚠ WARNING: During the stability test, there must be no movement whatsoever. If any movement occurs, repeat the procedure to adjust the lever pressure.

⚠ WARNING: The lever must never be pressed if the Stretcher set is not inserted inside the hole, since it will then damage the working of the Cranial Fixation System.

⚠ WARNING: The pressure of the set comes factory-adjusted.

NOTE: See chapter 9 for more information how to adjust the lever pressure.

6.2.2. ASSEMBLING THE HEADFIX BASE SET - HF051000FAA ON THE SURGICAL TABLE

1. First, remove the handle **c** from the tube, turning it counterclockwise;

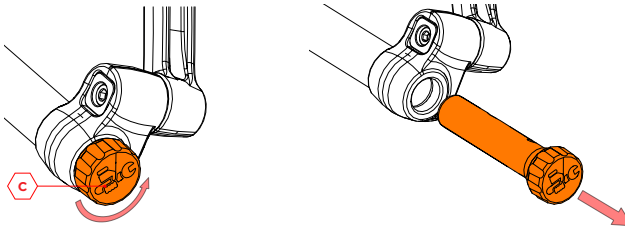
Figure 18

2. After removing the Base unit set's handle, first lift the hex key **I** so that it is perpendicular to the handle **II**;

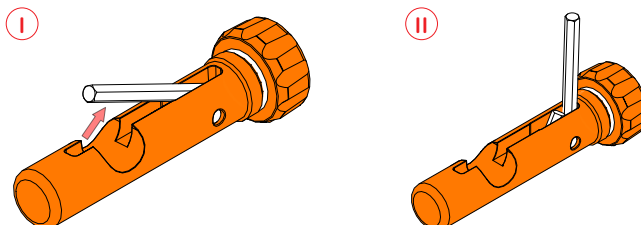
Figure 19

3. Insert the hex key into the connection lock screw (left and right) **I**, then turn the handle counter-clockwise until the arms are able to be adjusted **II**;

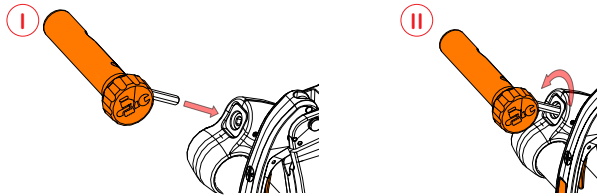
Figure 20

4. Adjust the distance between the centers of the axes according to the distance between the centers of the surgical table connections that will be used for the procedure;

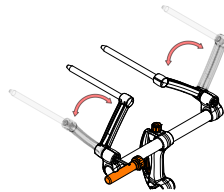
Figure 21

⚠ WARNING: The Base unit must be positioned on the surgical table in such a way that the set is symmetrical, leaving the tube parallel to the surgical table.

5. After adjusting the distance between the centers of the axes, lock the arm adjustment mechanism by turning the handle with the hex key clockwise;

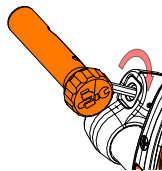
Figure 22

6. Return the hex key to its initial position;

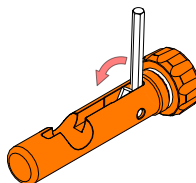
Figure 23

7. Return the handle on the Base unit set **I** and then turn it clockwise **II**;

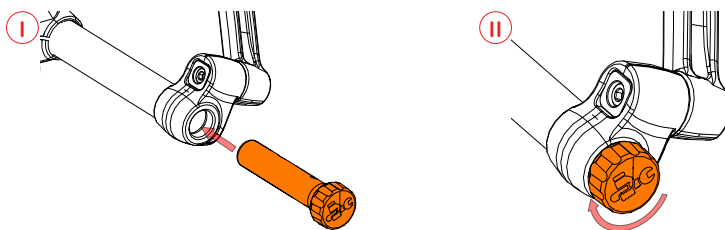
Figure 24

8. Press the lever's safety lock **d** **I** and, with the safety lock still pressed, pull the lever and leave it unlocked **II**;

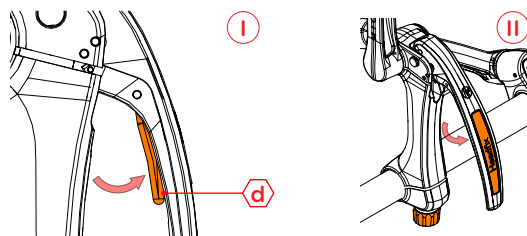
Figure 25

9. After the base assembly has all the joints released, leave it pointing toward the floor in order to facilitate the fixation of the articulated system by the user;

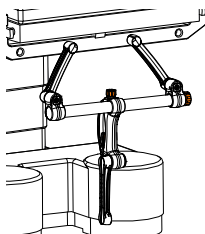
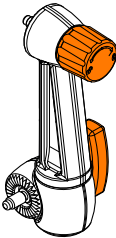
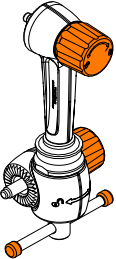
Figure 26

10. Lock the Base unit set using the fixation resource on the surgical table.

7. ARTICULATED SYSTEM


The articulated system connects the Base assemblies and the skull clamp. It is designed to provide flexibility in positioning the patient's head. Available in two models:

	ITEM	MODEL
C		<u>HF042000FAA</u> HEADFIX BASIC ARTICULATED SYSTEM
D		<u>HF052000FAA</u> HEADFIX ARTICULATED SYSTEM

NOTE: Both are compatible with the Headfix Basic Base (HF041000FAA), the Headfix Base (HF051000FAA), the Headfix Basic Skull Clamp (HF043000FAA), and the Headfix Skull Clamp (HF053000FAA).

7.1. HEADFIX BASIC ARTICULATED SYSTEM - HF042000FAA

The HeadFix Basic Articulated System allows 360° rotation and greater flexibility in patient positioning.

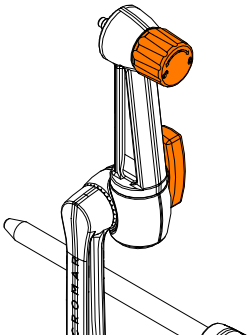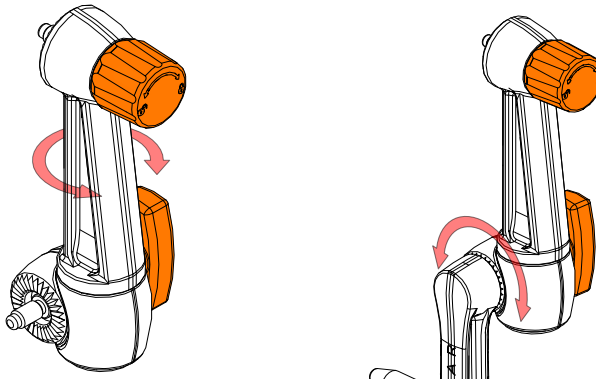
Figure 27

7.1.1. ASSEMBLING THE HEADFIX BASIC ARTICULATED SYSTEM - HFO42000FAA ON THE BASE SET

1. Insert the pin **a** of the Headfix Basic Articulated System into the base stretcher coupling **b** and adjust its position so that the teeth fit evenly.

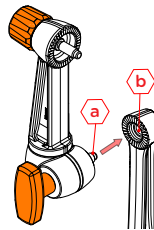
Figure 28

2. Turn the knob **b** clockwise to secure (I) the Headfix Basic Articulated System to the base in a way that maintains the 360° rotational movement (II).

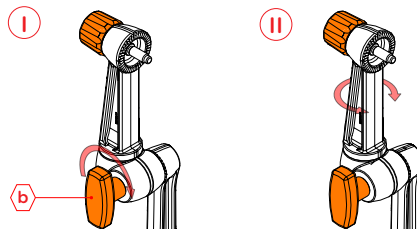
Figure 29

3. After fixing the articulated system to the base assembly, leave it pointing toward the floor in order to facilitate the user's fixation to the skull clamp;

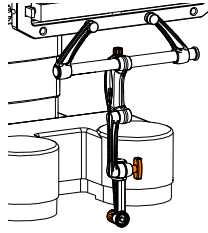
Figure 30

7.2. HEADFIX ARTICULATED SYSTEM - HF052000FAA

The HeadFix Articulated System features a pivoting mechanism that allows the equipment to tilt and rotate 360°, providing greater flexibility in patient positioning.

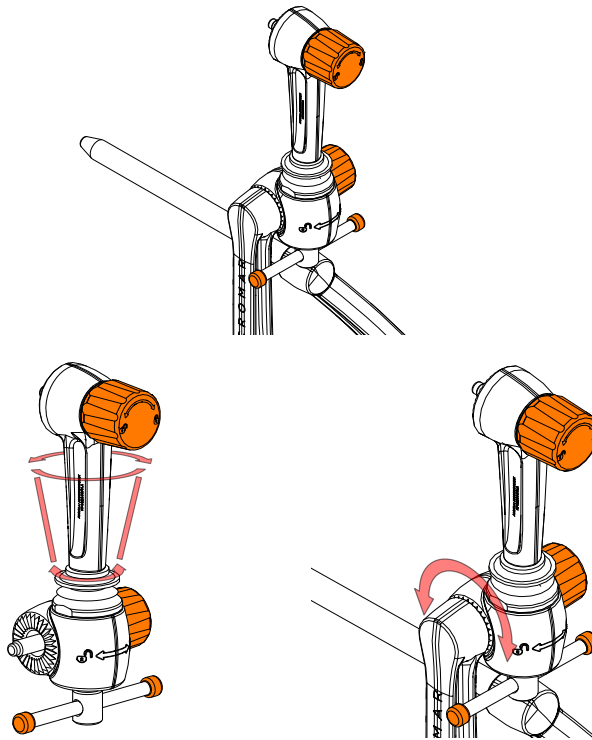
Figure 31

7.2.1. ASSEMBLING THE HEADFIX ARTICULATED SYSTEM - HF052000FAA ON THE BASE SET

1. Insert the pin **a** of the Headfix Articulated System into the coupling of the base stretcher **b** and adjust its position so that the teeth fit evenly.

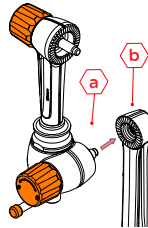
Figure 32

2. Turn the knob **c** clockwise (🔒) to secure the Headfix Articulated System to the base;

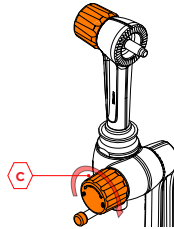
Figure 33

3. Pull the rod **d** **I**, then turn it clockwise (🔒) **II** to unlock the Headfix Articulated System;

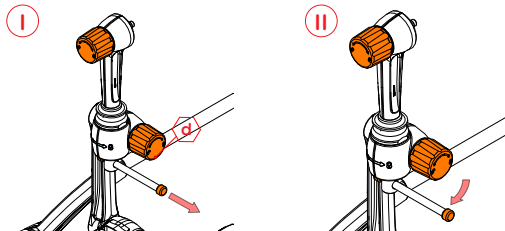
Figure 34

4. After fixing the articulated system to the base assembly, leave it pointing toward the floor in order to facilitate the user's fixation to the skull clamp;

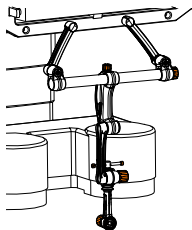
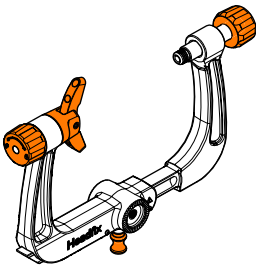
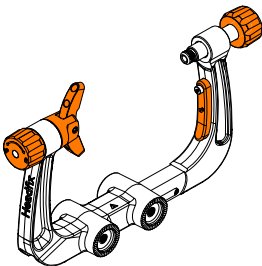
Figure 35

8. SKULL CLAMP

The skull clamp is an assembly that enables fixation of the patient's skull through the application of load distributed at three points around the patient's head. Available in two models:

ITEM		MODEL
E		HF043000FAA HEADFIX BASIC SKULL CLAMP
F		HF053000FAA HEADFIX SKULL CLAMP

NOTE: Both are compatible with the Headfix Basic Articulated System (HF042000FAA) and the Headfix Articulated System (HF052000FAA).

8.1. HEADFIX BASIC SKULL CLAMP - HF043000FAA

O The HeadFix Basic Skull Clamp has two integrated couplings that provide better positioning and facilitate the attachment of accessories. Its adjustment system allows opening for different head sizes, according to the distance illustrated in Figure 36.

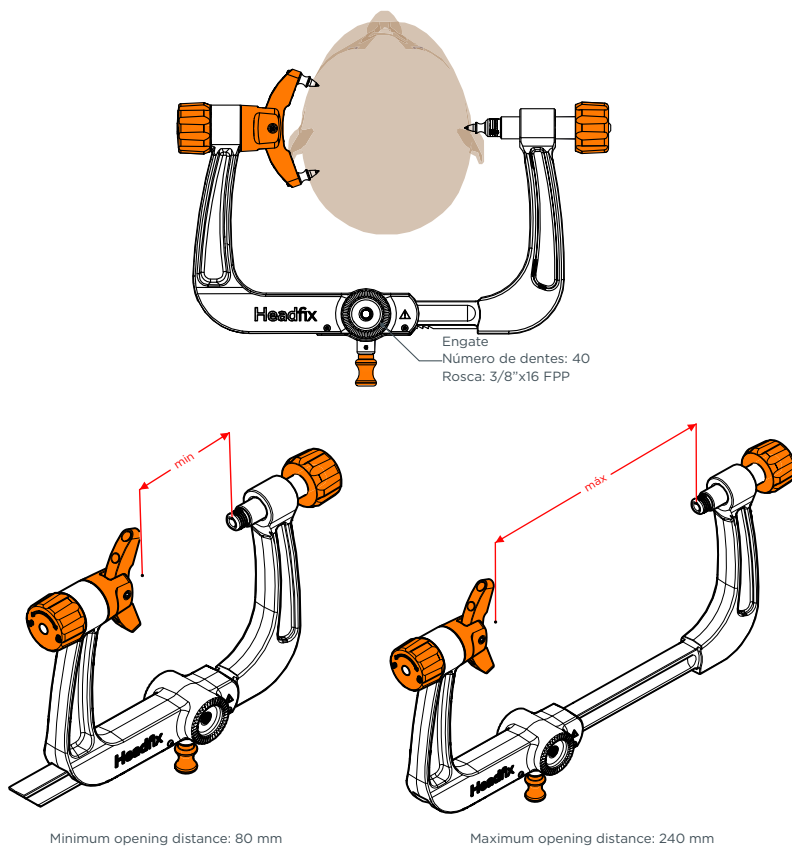
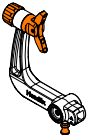
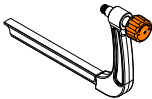
Figure 36

NOTE: For the entire skull clamp assembly process, assistance from another person is necessary to support the patient's head.

The Headfix Basic Cranial Clamp - HF043000FAA consists of 2 assemblies:

	ITEM	DESCRIPTION	QTY.
A		Fixed arm	1
B		Mobile Arm	1

8.1.1. POSITIONING THE HEADFIX BASIC SKULL CLAMP - HF043000FAA ON THE PATIENT’S HEAD

1. Pull the safety lock **a** of the fixed arm **I**, and then, while keeping the lock pulled, adjust the distance until it is possible to safely insert the patient’s head **II**;

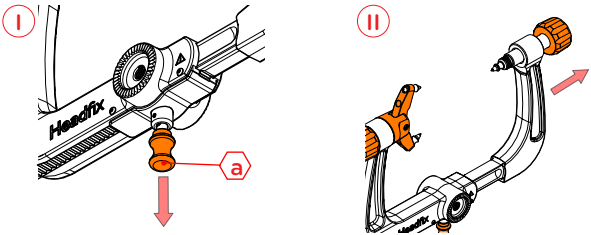
Figure 37

2. Position the skull clamp so that the fixation pins are aligned with the centerline of the patient’s head;

⚠ WARNING: For information about the assembly of the fixation pins, refer to chapter 8.3.

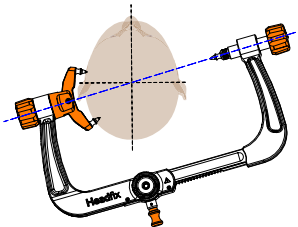
Figure 38

3. The pins inserted into the semiarc at the time of fixation must be equidistant from the pin inserted into the dynamometer, providing a symmetrical and more stable distribution of the load on the patient's head;

⚠ WARNING: For information on how to adjust the semi-arc, refer to chapter 8.4.

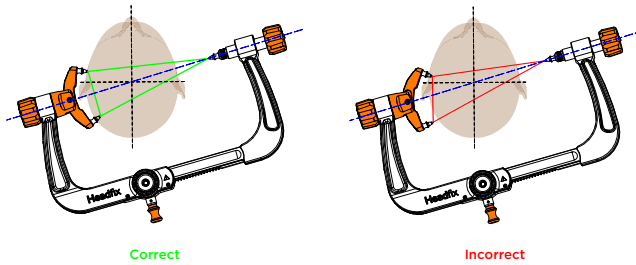
Figure 39

4. First, place the fixed arm against the patient's head (I), and then push the movable arm until the tip of the fixation pin touches the surface of the patient's head (II);

⚠ WARNING: For information about the operation of the dynamometer, refer to chapter 8.5.

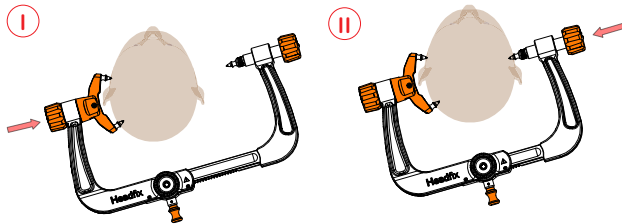
Figure 40

8.1.2. STABILIZING THE PATIENT'S HEAD

5. Insert the threaded pin (b) into the coupling of the skull clamp (c) and adjust its position so that the teeth fit evenly (I), then rotate the knob clockwise (II) to secure the cranial clamp to the articulated system;

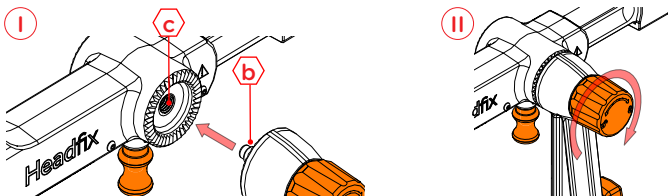
Figure 41

⚠ WARNING: Make sure that, at the time of fixation, the locking teeth are properly engaged.

6. Turn the knob **d** clockwise, locking the articulation of the articulated system;

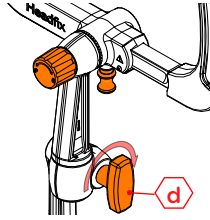
Figure 42

7. Adjust the position as needed for the surgical procedure **I**, then push the lever until the locking mechanism engages **II**;

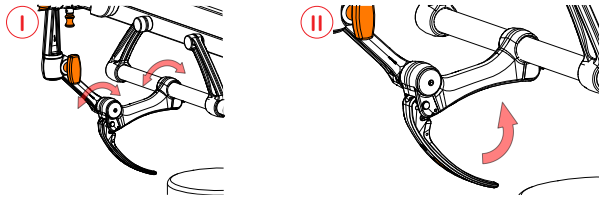
Figure 43

⚠ WARNING: The lever must never be pressed if the Stretcher is not inserted inside the hole, since it will then damage the working of the Cranial Fixation System.

⚠ WARNING: Make sure that at the time of fixing the locking teeth are correctly engaged.

⚠ WARNING: After activating the semiarc's locking mechanism (**🔒**), it's recommended to check if the lock actually happened.

⚠ WARNING: The patient's physical and pathological conditions must be taken into consideration prior to applying the load to fix the patient's head..

⚠ WARNING: The inappropriate adjustment of the Skull Clamp on the patient's head could cause the pins to slide, causing serious damage such as laceration of the scalp or falling of the head of the patient, bringing the risk of death.

8.2. HEADFIX SKULL CLAMP - HF053000FAA

The HeadFix Skull Clamp has four integrated couplings that provide better positioning and facilitate the attachment of accessories. Its adjustment system allows opening for different head sizes, according to the distance illustrated in Figure 44.

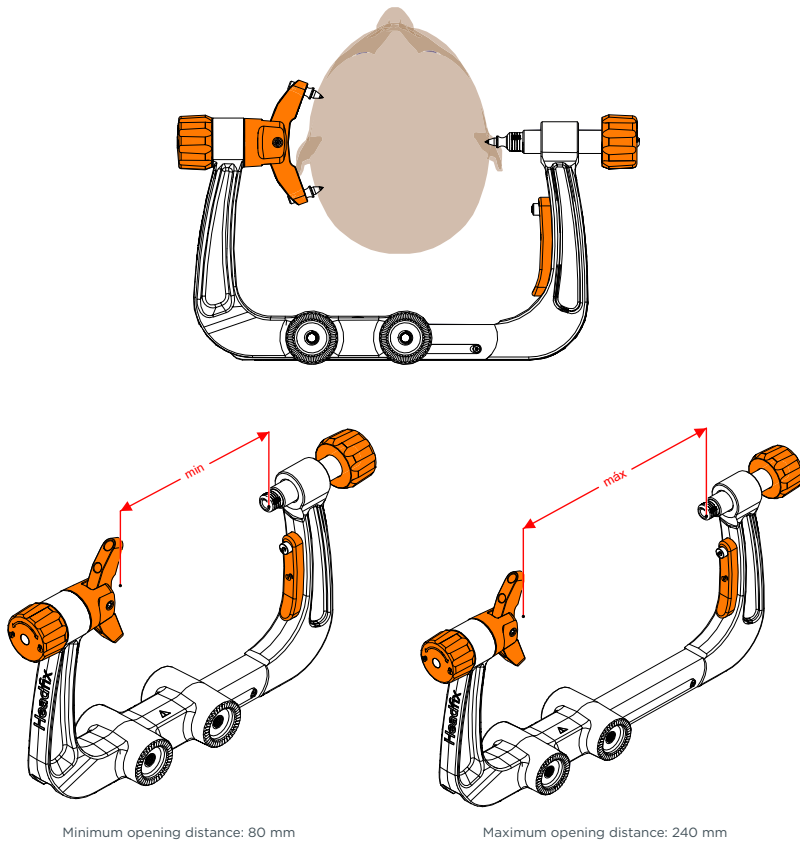
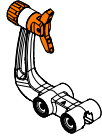
Figure 44

NOTE: For the entire skull clamp assembly process, assistance from another person is necessary to support the patient's head.

The Headfix Skull Clamp - HF053000FAA consists of 2 assemblies:

	ITEM	DESCRIPTION	QTY.
(A)		Fixed arm	1
(B)		Mobile Arm	1

8.2.1. POSITIONING THE HEADFIX SKULL CLAMP - HF053000FAA ON THE PATIENT'S HEAD

1. Press the safety trigger (a) of the fixed arm (I), then while holding it pressed, press the adjustment lock (b) and adjust the distance until it is possible to safely insert the patient's head (II);

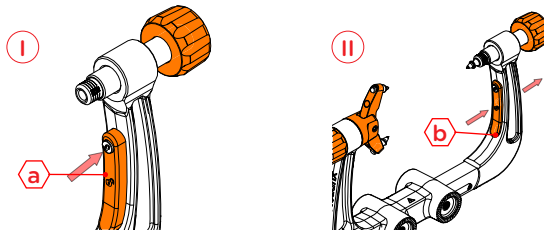
Figure 45

2. Position the skull clamp so that the fixation pins are aligned with the centerline of the patient's head;

⚠ WARNING: For information about the assembly of the fixation pins, refer to chapter 8.3.

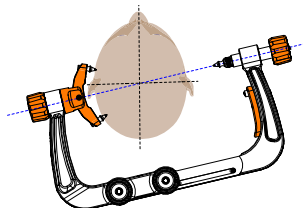
Figure 46

3. The pins inserted into the semicircle at the time of fixation must be equidistant from the pin inserted into the dynamometer, providing a symmetrical and more stable distribution of the load on the patient's head;

⚠ WARNING: For information on how to adjust the semiarc, refer to chapter 8.4.

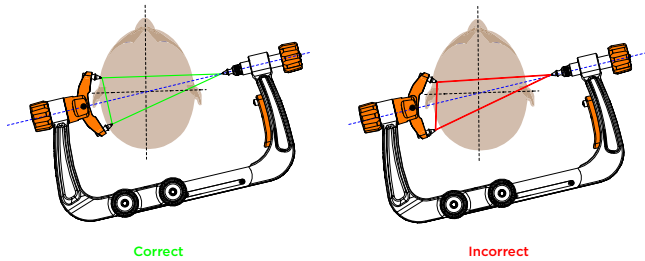
Figure 47

4. First, place the fixed arm against the patient's head **I**, then push the movable arm until the tip of the fixation pin touches the surface of the patient's head **II**:

⚠ WARNING: For information about the operation of the dynamometer, refer to chapter 8.5.

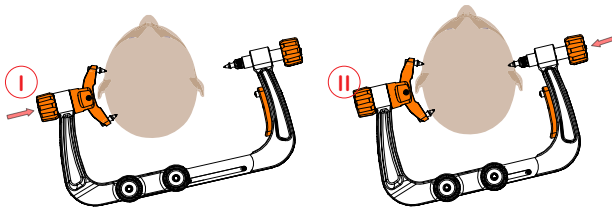
Figure 48

8.2.2. STABILIZING THE PATIENT'S HEAD

5. Insert the threaded pin into **c** the coupling of the skull clamp **d** and adjust its position so that the teeth fit evenly **I**, then rotate the knob clockwise (**b**) to secure the skull clamp to the articulated system **II**:

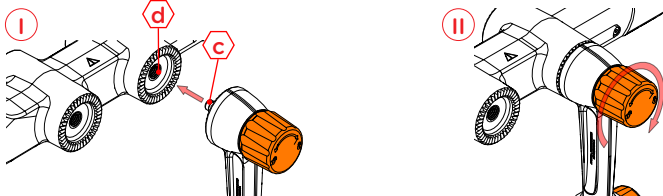
Figure 49

⚠ WARNING: Make sure that, at the time of fixation, the locking teeth are properly engaged.

6. Turn the knob **e** clockwise, locking the articulation of the articulated system;

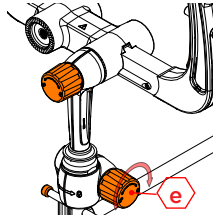
Figure 50

7. Adjust the position according to the needs of the surgical procedure **I**, then push the lever until the locking mechanism engages **II**;

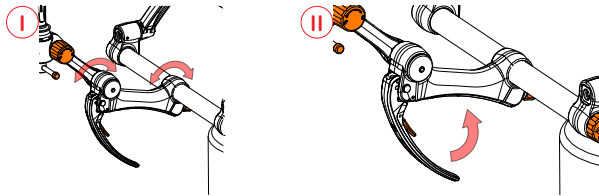
Figure 51

⚠ WARNING: The lever must never be pressed if the Stretcher is not inserted inside the hole, since it will then damage the working of the Cranial Fixation System.

⚠ WARNING: Make sure that at the time of fixing the locking teeth are correctly engaged.

⚠ WARNING: After activating the semiarc's locking mechanism (**II**), it's recommended to check if the lock actually happened.

⚠ WARNING: The patient's physical and pathological conditions must be taken into consideration prior to applying the load to fix the patient's head..

⚠ WARNING: The inappropriate adjustment of the Skull Clamp on the patient's head could cause the pins to slide, causing serious damage such as laceration of the scalp or falling of the head of the patient, bringing the risk of death.

8.3. ASSEMBLY OF FIXATION PINS ON THE SKULL CLAMP

The fixation pins (adult or pediatric) are products used together with the skull clamp to secure the patient's head.

8.3.1. PREPARATION FOR USE OF FIXATION PIN - ADULT MODEL

1. Insert a fixation pin with the protective cover into the dynamometer assembly;

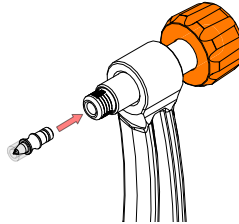
Figure 52

2. Insert the pins with the protective covers into the slots furthest from the center of the semiarc, ensuring that there is no gap between the pin and the semiarc;

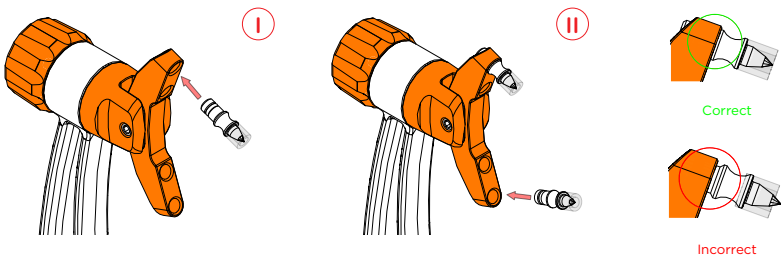
Figure 53

⚠ WARNING: The surgeon must evaluate the patient's condition, including their age, before planning the surgery.

NOTE: Before adjusting the cranial clamp on the patient's head, remove the protective covers from the fixation pins.

8.3.2. PREPARATION FOR USE OF FIXATION PIN - PEDIATRIC MODEL

1. Insert a fixation pin with the protective cover into the dynamometer assembly;

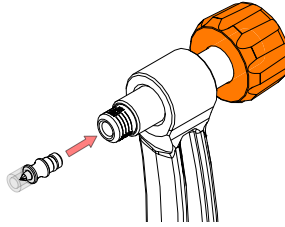
Figure 54

2. Insert the pins with the protective covers into the slots furthest from the center of the semiarc, ensuring that there is no gap between the pin and the semiarc;

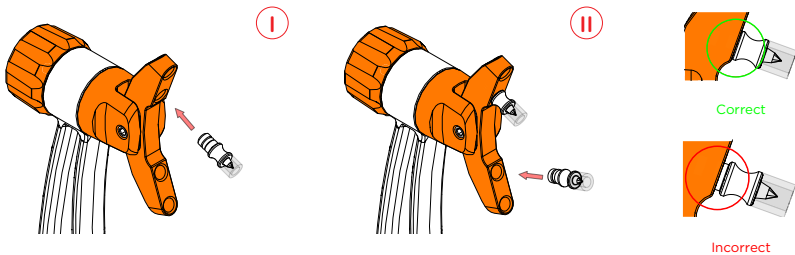
Figure 55

⚠ WARNING: The surgeon must assess the patient's condition, including their age, before planning the surgery.

NOTE: Before adjusting the skull clamp on the patient's head, remove the protective covers from the fixation pins.

8.4. SEMIARC

The semiarc allows the use of fixation pins in both adult and pediatric models, and features a system that offers 360° rotation and $\pm 22^\circ$ tilt for better adjustment and positioning of the head.

NOTE: The system and functioning of the semiarc are the same for both the HeadFix Basic Skull Clamp - HF-043000FAA model and the HeadFix Skull Clamp - HF053000FAA model.

⚠ WARNING: The surgeon must evaluate the patient's condition, including their age, before planning the surgery.

8.4.1. SEMIARC ADJUSTMENT

1. Rotate the semiarc handle counterclockwise (↺) to release the rotational movement;

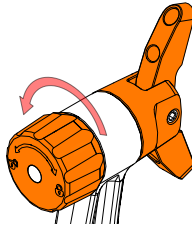
Figure 56

2. Adjust the position of the semiarc as needed for the surgical procedure;

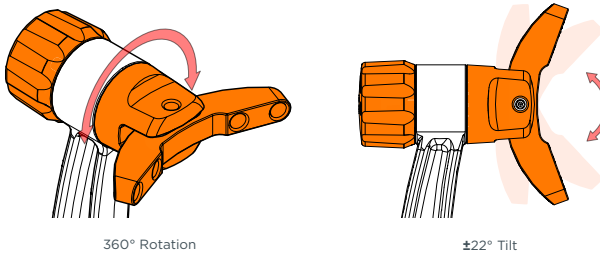
Figure 57

3. Rotate the semiarc handle clockwise (↻) until the mechanism that locks the adjustment movement is engaged.

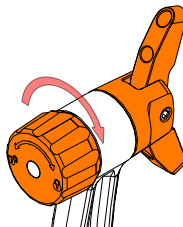
Figure 58

⚠ WARNING: After activating the semicircular arc locking mechanism (↻), it is recommended to verify that the lock has actually engaged.

⚠ WARNING: Only rotate the semicircular arc when the locking mechanism is in the unlocked position (↺); otherwise, the internal locking mechanism will be damaged.

8.5. DYNAMOMETER

The dynamometer is a mechanism that allows the application and visualization of the load in Newtons (N) at three points on the front and two points on the back.

NOTE: The dynamometer mechanism is the same for both the HeadFix Basic Skull Clamp - HF043000FAA model and the HeadFix Skull Clamp - HF053000FAA model.

8.5.1. DYNAMOMETER FUNCTIONING

1. Rotate the dynamometer handle clockwise (➡) to apply the load;


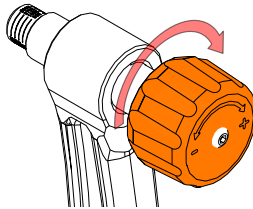
Figure 59

2. Adjust the necessary load to secure the patient's head, controlling it through the front (a) and rear (b) indicators;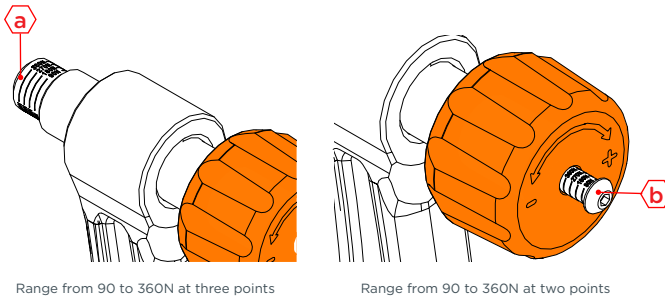

Range from 90 to 360N at three points

Range from 90 to 360N at two points

Figure 60

3. Rotate the dynamometer handle counterclockwise (⬅) to remove the applied load;

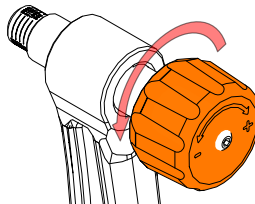
Figure 61

⚠ WARNING: Before applying the load to secure the patient's head, the patient's physical and pathological conditions must be taken into account.

⚠ WARNING: Improper adjustment of the skull clamp on the patient's skull may cause the pins to slip, resulting in serious injuries such as scalp lacerations or the patient's head falling, posing a risk of death.

9. ADJUSTMENT OF LEVER PRESSURE

⚠ WARNING: Check the lever tension at regular intervals.

⚠ WARNING: Adjustment of lever pressure must only be done when there is a real need to do so, since lever pressure already comes factory-adjusted.

1. Pull the lever and leave it unlocked;

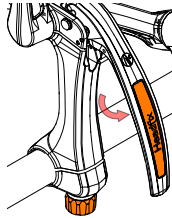
Figure 62

2. Insert the equipment's hex key into the hexagonal profile of the lever (quick lock) **I**, then rotate it counterclockwise to release the lever pressure adjustment movement **II**.

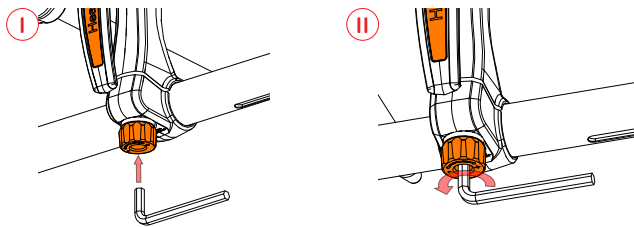
Figure 63

3. Hold the lever fully lifted **I**, then manually rotate the handle clockwise to apply more pressure (+) or counterclockwise to apply less pressure (-) **II**.

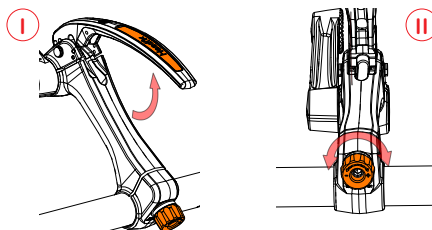
Figure 64

⚠ WARNING: After adjusting the lever pressure, make sure the lever movement is smooth but firm.

4. After adjusting the lever pressure, press the lever until it locks **I**, then insert the hex key into the hexagonal profile of the lever (quick lock) and rotate clockwise to lock the lever pressure adjustment movement **II**.

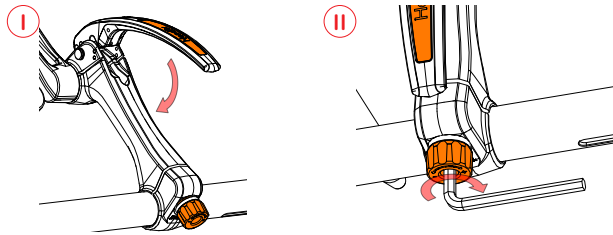
Figure 65

⚠ WARNING: The lever must never be removed from the set;

⚠ WARNING: If the Extender assembly is not mounted on the Lever assembly (Quick lock), the lever must never be pressed, as this may interfere with the equipment's operation.

⚠ WARNING: Ensure that the lever's support profile is properly placed under the support pins.

⚠ WARNING: If the lever is not connected to the support pins, carefully reinsert it, ensuring that the backrest profile of the lever is correctly placed on the support pins.

⚠ WARNING: Excessive pressure on the lever could damage the locking mechanism.

10. DISASSEMBLY

After the surgery is completed, follow the steps below for the proper disassembly of the HeadFix Cranial Fixation System:

1. Disconnect the Cranial Fixator from the Articulated System;
2. Rotate the Dynamometer handle counterclockwise () to remove the applied load;
3. Engage the safety lock and then adjust the opening distance of the movable arm until it is possible to safely remove the patient's head;
4. Remove the Cranial Fixator locking pins;
5. Disconnect the Articulated System from the Base;
6. Remove the Base from the surgical table;
7. Send all components used for cleaning as specified in this manual.

⚠ WARNING: For more information on the correct cleaning procedure, refer to Chapter 11.

11. CLEANING

The table below presents the cleaning methods for the HeadFix Cranial Fixation System:

Cleaning Method	
Manual Cleaning (immersion)	Immerse the disassembled parts in a container with cold to warm demineralized water (up to 40°C) with enzymatic detergent (dilution as recommended by the manufacturer). Allow the solution to act for at least 15 minutes. With a soft brush remove the residue from the items, taking care of the recesses, threads and lumens. Rinse with running demineralized water. Dry with surgical compress or wipe and clean compressed air.
Cleaning Automated (Ultrasound)	Immerse the disassembled parts in an ultrasonic washer tank with cold to warm demineralized water solution (up to 40°C) with enzymatic detergent (dilution as recommended by the manufacturer). Operate the ultrasound for at least 15 minutes. Rinse with running demineralized water. Dry with surgical compress or wiper and clean compressed air.

-  **WARNING:** Do not sterilize. The components may be damaged by sterilization.
-  **WARNING:** Items exposed to Creutzfeldt-Jakob disease should be discarded according to local regulations;
-  **WARNING:** If the parameters for cleaning and sterilization are not followed as described herein the contamination control may not be guaranteed or damage may be caused to the product;
-  **WARNING:** Do not use abrasive and corrosive agents for cleaning, since they can cause damage to the product;
-  **WARNING:** After cleaning, the components should be immediately dried using a dry compress and compressed air. Pay attention to recesses and screws;
- NOTE:** Follow the manufacturer instructions for the washing machine;
- NOTE:** Follow the information specified by the detergent manufacturer.

12. PREVENTIVE AND CORRECTIVE MAINTENANCE

To guarantee the appropriate functioning and performance of the system, it must return to preventive maintenance **after 18 months**.
If there is any defect or malfunction before 18 months have passed, do not use and immediately contact Micromar or your authorized sales representative.
Micromar is the only authorized technical service provider for the Headfix Cranial Fixation System.
After the expiration of the 18-month period, it's need sent to Micromar, and the **every 14 months**, for assessment and to perform any maintenance or disposal.

PREVENTIVE MAINTENANCE TABLE			
____/____/____ Quality control	____/____/____ Quality control	____/____/____ Quality control	____/____/____ Quality control
1st Preventive maintenance	2st Preventive maintenance	3st Preventive maintenance	4st Preventive maintenance

13. SYMBOLS

Symbols	Subtitle
	Keep away from sunlight/Keep away from heat
	Manufacturer
	Attention: Consult the instructions for use
	Keep dry/Keep away from humidity
	Consult the instructions for use
	Date of manufacture
	Serial number
	Stacking limit
	Fragile
	This way up
	Lot number

14. TRANSPORT AND STORAGE

• TRANSPORT

- For transportation of the HeadFix Cranial Fixation System, all parts must be properly placed in their designated locations.
- During handling and transportation, avoid actions that may damage the packaging, as well as drops or impacts that could affect the product's functionality.
- The maximum recommended stacking for transportation is three boxes.

• STORAGE

- Store in a dry, cool, odorless environment, protected from direct sunlight and heavy dust.
- The maximum recommended stacking for storage is three boxes.
- Before using the product, the packaging must be inspected for integrity to ensure it has not been compromised.
- Do not use if the packaging is damaged and/or tampered with.

 **WARNING:** *The product must be transported and stored in the packaging provided by Micromar.*

15. WARNINGS AND PRECAUTIONS

- Accidental movements involving cranial fixation system components during the procedure can compromise the efficacy of the procedure.
- Avoid dropping, twisting, crushing, impacting or any action that will deform the product.
- In the event that one of the above occurs, the user should contact Micromar for assessment of the product.
- Incorrect reading of equipment scales (user error in reading or because of poor equipment condition) can put the patient's life at risk.
- Screw in and correctly attach the movable parts.
- To perform the procedure, the system must be clean in accordance with the instructions in this manual.
- Immediately after the procedure is completed, the system must be cleaned in accordance with the instructions in this manual.
- This product is solely made for use in surgical centers.
- This product may only be manipulated by a surgeon specialized in neurosurgery with experience performing cranial fixation.
- The surgeon must consider the patient's physical and pathological conditions when securing the cranial fixation pin together with the product to the patient's skull, in order to avoid causing harm.
- Failure to follow the instructions contained in the user manual provided by Micromar may result in incorrect use of the HeadFix Cranial Fixation System, putting the patient's life at risk.
- Use the maintenance service provided by Micromar or an authorized representative to correctly repair the system when there are any problems with its functioning.
- For the repair or replacement of any component, contact Micromar through customer service or any authorized representative.
- The Headfix Cranial Fixation System used to treat patients with Creutzfeldt-Jakob disease MUST NOT be reused on other patients.
- Disposal of the product should be done according to local laws.
- The HeadFix Cranial Fixation System has a useful life of 5 years from the date of issuance of the invoice.

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