Símbolos / Symbols / Símbolos

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ENGLISH

Product Description: The Disposable Fiducial Marker for Navigation is a single-use product composed of a hypoallergenic adhesive, a transparent cover made of low-density polyethylene, filled with a natural polymer. It has a set of eight units, used in image-guided surgical navigation procedures.

Intended Use: The Disposable Fiducial Marker for Navigation is a single use device which aims to provide fiducial references for the registration based on markers for subsequent surgical image-guided navigation procedure.

Purpose and Working Principle: The Disposable Fiducial Marker for Navigation is used in conjunction with a Surgical Navigator, which is dedicated to guiding surgeons, through images, during surgical procedures. It allows surgical instruments to be spatially tracked and visualized in sync with the patient's anatomical structures in real time. The technique, known as Image-Guided Surgery, consists of having the patient's anatomy and surgical instruments reproduced through graphic-computational reconstructions using monitors.

The Disposable Fiducial Marker for Navigation is used to mark reference points in patient coordinate space and image coordinate space. Fiducials consist of a circular structure with a geometric center tangible by a tip, are identified in CT and MRI images and have adhesive properties that guarantee adherence to the patient's anatomical structure.

Caution:

The Disposable Fiducial Marker for Navigation is intended for specialized procedures, which must be performed by duly qualified professionals. The product must be used in a surgical environment and under conditions suitable for the health and safety of the patient.

How to Use the Disposable Fiducial Marker for Navigation:

1. Visually check the primary packaging (blister sealed with Tyvek);
2. Open the blister;
3. Remove the markers from the blister;
4. Define the markers localization on the patient;
5. Peel off the protective backing;
6. Place the markers pressing them on the patient;
7. Rub firmly to ensure adhesion;
8. Accomplish the CT and/or MRI exams;
9. Export the CT images and/or MRI to the Navigator;
10. Accomplish the registration pivoting the instrument in the conical holes of the markers;
11. Accomplish the surgical procedure;
12. Remove the markers from the patient;
13. Discard the markers.

Cautions / Warnings:

- Before using the product, check the packaging regarding its integrity in order to ensure that the product cleaning is not compromised.
- Do not use the product if the packaging is damaged.
- Check the expiration date of the product before opening the packaging.
- Avoid any action that will deform the product, because it can compromise its cleaning and its operation.
- This product is intended to be used only in the room where the CT images and MRI are acquired and in surgical center.
- The fiducial markers must only be used by a physician specialized in neurosurgery and trained by Micromar or with experience in navigation surgery.
- The markers must only be used with Micromar Navigator and instruments. The use of any part, accessory or material not specified is entirely the user's responsibility.
- The product is supplied non-sterile. The fiducial markers must not be cleaned or sterilized by the user.
- The markers are single use devices, therefore, it is forbidden to reprocess it.
- Make sure that the markers do not obstruct the posterior skull fixation by the Cranial Fixation Equipment.
- Make sure that the surgeon access to the markers is not obstructed considering the patient position on the surgery table.
- Make sure that the markers are arranged asymmetrically to the median sagittal to avoid ambiguity.
- The markers arrangement is specific for each patient and for each procedure and it is necessary to use at least 5 markers.
- Do not replace and/or remove the markers from the patient after the images acquisition and before the registration because their position is essential for proper registration.
- The fiducial markers may be substituted by another only before the images acquisition.
- The excess of hair and skin oiliness can impair the adhesiveness of the markers.
- The markers must be placed in intact and dry skin.
- The fiducial markers are not sterile, therefore, in sterile field, they must be protected so they do not contact the sterile instruments and any other sterile parts.
- Failure to follow this instruction for use and the associated use equipment manual can lead to incorrect use of the device and therefore misuse, which increases the risk of death to the patient.

Contraindications: Do not use the product in the risk situations foreseen by the surgeon.

Adverse effects: No side effect was found when the product is used by a specialized professional following this instruction for use.

Disposal  : Dispose of the product in accordance with current hospital regulations and local legislation.

Traceability: All Micromar products have sequential batches that allow traceability, thus promoting greater safety for professionals qualified for the procedure. Through this batch number it is possible to know the entire history of the product from the manufacturing process to the moment of distribution.

Storage: The Disposable Fiducial Marker for Navigation must be stored in a dry and cool place, at a temperature of 20 to 30° C, with relative humidity between 55 to 75%. It must be protected from direct solar radiation, in its original, unopened packaging and must not be damaged.

Form of Presentation: The product is for single use with the following packaging:

- Primary packaging: blister made of heat-sealed PET with Tyvek containing 08 markers;
- Secondary packaging: surgical grade paper + PET/PP film;
- Tertiary packaging: triplex paper box.

Expiration date: The shelf life according to the stability study is 4 years; An expiration date once opened does not apply.

Additional Information: Single-use product. In an incident caused by the product occurs, the professional must immediately inform the manufacturer.

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