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ENGLISH

Product Description: The Disposable Screw for Cranial Fixation is a single-use product, its composition is aluminum, available in different sizes and sets. It is used in stereotactic neurosurgical procedures.

Intended Use: The Disposable Screw for Cranial Fixation is a single use device which aims to enable the fixation of the patient head during surgical procedures. It is used in stereotactic neurosurgical procedures.

Purpose and Working Principle: When coupled to the Micromar Stereotactic System, the Disposable Screw for Cranial Fixation is pressed against the points established on the patient's head until it is fixed to the cranial bone.

 Caution:

The Disposable Screw for Cranial Fixation is intended for specialized procedures, which must be performed by duly qualified professionals. The product must be used in a surgical environment and under conditions suitable for the health and safety of the patient.

How to use Disposable Screw for Cranial Fixation:

1. Open the surgical grade packaging;
2. Visually check the primary packaging (blister sealed with Tyvek);
3. Open the blister;
4. Remove the screws from the blister;
5. Mount the screws in the Stereotactic System;
6. Fasten the screws to the patient's skull using a 4 mm allen screwdriver;
7. Accomplish the surgical procedure;
8. Remove the screws from the Stereotactic System;
9. Discard the screws.

Cautions / Warnings:

- Before using the product, check the packaging regarding its integrity in order to ensure that the product sterility is not compromised.
- Do not use the product if the sterilization packaging is damaged.
- Check the expiration date of the product before opening the packaging.
- Avoid any action that will deform the product, because it can compromise its sterility and its operation.
- This product is intended to be used only in surgical center and in the room where the CT images are acquired.
- This product must only be used with Micromar Stereotactic System. The use of any part, accessory or material not specified is entirely the user's responsibility.
- The Disposable Screw for Cranial Fixation (4 pediatric) - Aluminum - PD0004FAA is recommended for children between 3 and 9 years old. From 10 years old, it is possible to use the Disposable Screw for Cranial Fixation PD0005FAA, PD0006FAA and PD0010FAA. Nevertheless, the physician must evaluate the patient's condition, considering his / her age, before planning the surgery. When possible, avoid positioning the screws over abnormally thin bone or in areas near prior craniotomy bone flap sites. The physician must consider the skull condition, including its thickness, before planning the surgery.
- The screws have a sharp tip, therefore they must be handled carefully.
- This product must only be used by a physician specialized in neurosurgery with experience in stereotactic surgery.
- The product is supplied sterilized by Ethylene Oxide (EO).
- This product is a single use device, therefore, it is forbidden to reprocess and/or re-sterilize it.
- The surgery precision depends on the target coordinates calculation, the trajectory definition and the Stereotactic System assembly and operation performed by the surgeon.
- Failure to follow this instruction for use and the associated use equipment manual can lead to incorrect use of the device and therefore misuse, which increases the risk of death to the patient.
- The screws generate artifact in CT images, therefore do not position the screws near the desired target.

Contraindications: Do not use the product in the risk situations foreseen by the surgeon.

Adverse Effects: No side effect was found when the product is used by a specialized professional following this instruction for use.

Disposal : The screws are sharp pointed. Dispose of the product in accordance with current hospital regulations and local legislation.

Traceability: All Micromar products have sequential batches that allow traceability, thus promoting greater safety for professionals qualified for the procedure. Through this batch number it is possible to know the entire history of the product from the manufacturing process to the moment of distribution.

Storage: The Disposable Screw for Cranial Fixation must be stored in a dry and cool place, at a temperature of 20 to 30° C, with relative humidity between 55 to 75%. It must be protected from direct solar radiation, in its original, unopened packaging and must not be damaged.

 Presentation and Sterilization Metod:

The Disposable Screw for Cranial Fixation is supplied sterile by Ethylene Oxide (ETO) and for single use with the following packagind:

- Primary packaging: blister made of heat-sealed PET with Tyvek containing, PD0004FAA: 04 pediatric screws; PD0005FAA: 04 medium screws and 01 small screw; PD0006FAA: 04 large screws and 01 small screw; PD0010FAA: 04 large screws, 04 medium screws and 01 small screw.
- Secondary packaging: surgical grade paper + PET/CP;P film;
- Tertiary packaging: triplex paper box.

Expiration date: The shelf life according to the stability study is 4 years; An expiration date once opened does not apply.

Additional Information: Single-use product. If an incident caused by the product occurs, the professional must immediately inform the manufacturer.

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